Guldmann™

SR 801

DK Digitalvægt	
GB Digital Scale	
D Digitalwaage	
S Digitalvåg	
U.S.A. and countries outside the EU 22	(













GB SR 801 Digital Scale

Item no. 11340

1.00 Purpose and Use	8
1.01 Manufacturer	8
1.02 Description	8
1.03 Important/warning	
2.00 Description of Functions	8
2.01 Labels	
2.02 Operating SR 801	
2.03 More information	
3.00 Transport	10
4.00 Maintenance and storage	
4.01 Cleaning the SR 801	
4.02 Storing the SR 801	
5.00 Battery	10
6.00 Technical Specifications	10
7.00 EC Declaration of conformity	11

1.00 Purpose and Use

1.01 Manufacturer

V. Guldmann A/S Graham Bells Vej 21-23A DK - 8200 Århus N Tel. + 45 8741 3151

Tel. + 45 8741 3151 Fax. + 45 8741 3131

1.02 Description


The SR 801 Digital Scale is a compact precision scale designed specially for use with mobile and ceiling mounted hoists for disabled persons. Completely self-contained, the SR 801 combines the latest in precision strain gage technology and microprocessor signal conditioning to provide stable, accurate and repeatable weight data with a calibrated accuracy of 0,1% One button Auto-Zero Electronics makes patient weighting quick and easy.

1.03 Important/warning

- · Read the user manual fully before using the SR 801.
- Do not exceed the max. load of 270 kg/600 lbs.
- Do not use SR 801 where there is a risk of water coming into direct contact with it.
- · The display is not impact-proof, do not drop it.

2.00 Description of Functions

2.01 Labels on the SR 801

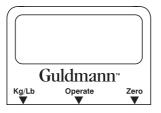




2.02 Operating SR 801

Note:

Readjust the weight indication between kg and lbs by removing the black plug as illustrated. With a screwdriver change the position of the small black switch that becomes visible.

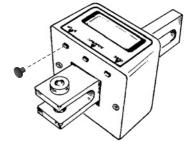


 Attach the top hook of the SR 801 Scale to the boom or the lifting strap of the patient lifter. Make sure that the hook is fully engaged, and the safety screw is properly installed.

Caution:

Tighten Safety Screw until fully engaged.


Attach the Lifting hanger to the bottom of the scale system and then attach the selected Sling to the Lifting hanger.

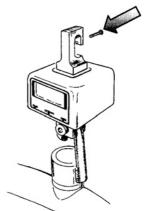

- With the sling in place, press the "ZERO" button to set the system zero. The digit "0" will pan across the display for a few seconds then the display will read "0.0".
- Remove the sling, position the patient in to the sling, then reattach to the Lifting hanger.
- Carefully lift the user. When the sling carrying the user is steady and freely suspended, press "Operate" to display the current weight.

Important

Always Re-Zero the scale before weighting.

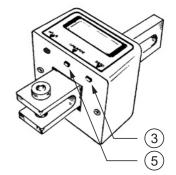

Note:

The display will automatically turn itself off after approx. 60 seconds to conserve battery power. If the display turns off before you are able to view the weight data, simply press the "Operate" button once again.

6. Carefully lower the user after reading the weight.

Caution:

This device is intended to obtain patient weight during transfer (i.e.: bed to chair). This device is NOT intended for use during patient transport and should be removed in such case.

2.03 More information

If you should have any questions or comments regarding the operation of the Guldmann Digital Scale, model SR 801 please contact the distributor.

3.00 Transport

Guldmann recommends that SR 801 is transported in its original packaging.

4.00 Maintenance and storage

4.01 Cleaning the SR 801

SR 801 can be cleaned using a damp sponge.

Do not use chemicals or autoclaving to cleanse the SR 801.

4.02 Storing the SR 801

The SR 801 should be kept in a dry room.

5.00 Battery

The SR 801 Digital Scale is powered by a single 9 volt battery. This battery should provide approximately 3000 readings before needing replacement. When battery replacement is needed a "B" will appear on the display. Simply remove the two screws on the case to access the battery.

Note:

The load cell contains no user serviceable components and should be serviced by autorized personel only.

Any unauthorized tampering will void the warranty/guarantee.

6.00 Technical Specifications

Weight range
Display type LCD
Display resolution
Display accuracy
Size (H x L x W)
Weight 0.450 kg/1 lb
Weight
Battery 9 Volt alkaline

Guldmann[™]

EC-DECLARATION	OF CONFO	RMITY	
Manufacturer	V. Guldmann A/S Graham Bells Vej 21-23A DK-8200 Aarhus N EAN country code: 57 EAN distribution No.: 07287 Phone +45 8741 3151 Fax +45 8741 3131		
Representative	Company		
	Address		
	Country		
	Phone		
Hereby declare that	Product	SR 801 Digital Scale	
	Type No.	XXXXX-YYYY-ZZZZ x)	
	x) XXXXX: Ref	erence No. of the product, YYYY: Serial No. of the product in the production year, ZZZZ: Production year	
	was manu	factured in conformity with the provisions in:	

Skejby 06.01.2003

• EN 50082-1: Electromagnetic compatibility
• EN 55022: Limits and methods of measurement of radio disturbance

according to the Council Directive 93/42/EEC of 14 June 1993 and 90/384/EEC

characteristics of information technology equipment

Place and day of issue

of 20 June 1990

Product responsible Technical Manager

U.S.A. and countries outside the EU

A. USERS GUIDE Digital Scale

Please read all warranty information and this entire chapter before use of the product.

B. WARRANTIES

The Guldmann lifting equipment is designed to be used for the lifting, transferring, and transporting of persons who are physically challenged or incapacitated and who are not able to physically self transfer.

If the product is used in an irresponsible manner or for any use other than that described above or in the enclosed written instructions, the manufacturer's warranty will be nullified and rendered void.

There are no warranties which extend beyond the description in the enclosed written instructions.

Guldmann warrants that its lifting equipment is free from defects in materials and workmanship under normal use. Guldmann warrants that the lifting equipment itself will perform substantially in accordance with the specifications set forth in the documentation provided with the equipment.

The above express warranties are made for a period of 365 days from the date the lifting equipment is delivered to you as the first user.

Your distributor will replace any lifting equipment which proves defective in materials or workmanship, without additional charge, on an exchange basis.

Your distributor will either replace or repair without additional charge any Guldmann lifting equipment that does not perform in substantial accordance with the specifications of the documentation.

Guldmann does not warrant that the functions contained in the lifting devices will meet your requirements or that the operation of the services will be uninterrupted or error-free. The warranty does not cover any of the part of the lifting equipment which has been subject to damage or abuse by you. The warranty does not cover any part of the lifting equipment which has been altered or changed in any way by you or others. Guldmann is not responsible for problems caused by changes in the operating charasteristics of the operating system which are made after the delivery of the lifting equipment. Any implied warranties including any warranties of merchant ability or fitness for a particular purpose are limited to the terms of the express warranties.

Guldmann shall not in any case be liable for special, incidental, consequential, indirect or other similar damages arising from any breach of these warranties even if Guldmann or its agent has been advised of the possibility of such damages.

You must call Guldmann or your distributor for an authorization to return any defective item during the warranty period. If your distributor is unable to correct your problem by telephone, you will be provided with a return authorization number and address for returning the defective item for warranty service or replacement. You must insure any defective item being returned as Guldmann does not assume the risk of loss or damage while in transit. Do not return items or warranty service to Guldmann.

The warranties set forth above are in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies.

Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann and should not be relied upon as such. The warranty gives you specific legal rights, and you may also have the other rights which vary from state/country to state/country.

Guldmann™











Time to care

V. Guldmann A/S

DK-8200 Århus N

Graham Bells Vej 21-23A

+45 8741 3100

Head Office:

Tel.

Guldmann GmbH

Söhnleinstr 8 D-65201 Wiesbaden Tel. +49 611 974 530 Fax +49 611 300 018 E-mail info@guldmann.de www.guldmann.de

Guldmann Sverige AB

Smålandsgatan 4 S-441 57 Alingsås Tel. +46 0322 55290 +46 0322 55320 E-mail info@guldmann.se www.guldmann.se

Guldmann Inc.

5505 Johns Road Suite 700 Tampa, FL 33634 Tel. 800 664 8834 813 880 0619

Tel.

Fax +45 8741 3131 E-mail info@guldmann.com 813 880 9558 www.guldmann.com E-mail info@guldmann.net